



Samarit products are designed to facilitate moving & handling using less effort. This makes work easier and prevents back injuries to the carers.

SAMARIT NURSING AIDS

For Rehabilitation and Homecare



GLIDEBOARDS

Page 2



THERAPEUTIC BED TOPPERS

Page 4



MOVING AND HANDLING SLINGS

Page 5



TURNPLATE, TURN CUSHION

Page 6



CAREGLIDE TRANSFER SHEETS

Page 7

High quality standards
SAMARIT transfer aids have:

- Design patents
- 2 years warranty



www.samarit.com

SAMARIT
safe patient moving & handling

Samarit transfer systems



Gap < 30 cm / 12 inch, SWL* = 150 kg
Gap < 20 cm / 8 inch, SWL* = 180 kg

Sliding boards

SAMARIT sliding boards are Nursing aids for **moving a seated patient** with limited forces **from the middle or edge of the bed to a wheelchair, a car seat or a toilet**. Here the patient may not be able to stand but can change position when sitting independently or with the help of a healthcare professional.

While the smooth surface reduces friction resistance to a minimum, non-slip adhesive strips ensure that the GLIDEBOARD doesn't slip away. The GLIDEBOARD is made of an **unbreakable** material, is **acid-resistant** and **easy to clean** and **disinfect**. A non-slip adhesive strip enables it to be positioned safely on the bed, wheelchair, car seat or other seating options. **Five versions** are available for different purposes.

The SAMARIT GLIDEBOARD range has been developed for special use with the support of therapists.

GLIDEBOARD

SLIDING BOARD / SEATED TRANSFER



CURVED GLIDEBOARD THE GREAT THE BIGGEST SELFGLIDE TETRAGLIDE

- Unbreakable
- Sliding surfaces
- Non slide sticker
- Lightweight
- Left/right usable
- Instruction for use
- CE mark
- 2 years warranty

Greater mobility with GLIDEBOARD

Seated transfer for care stations, therapy centers and home care.



THE BIGGEST

Order No. 476.0102

UDI/GTIN 7640144460921

This sliding board makes it easier for **healthcare professionals** to **transfer patients** from the **middle of the bed** to a **wheelchair**, to the **toilet**, to the **car seat** or a **chair**.

Size: 100 x 34 cm / 39 x 13 inch

SWL*: 220 kg / 485 lbs

Weight: 2.9 kg / 6.4 lbs

Material: HDPE



THE GREAT

Order No. 476.0080

UDI/GTIN 7640144460938

This sliding board makes it easier for **healthcare professionals** to **transfer patients** from **edge of the bed** to a **wheelchair**. The product is used mainly today to transfer patients **from wheelchairs into large vehicles** (SUVs etc.).

Size: 80 x 33 cm / 32 x 13 inch

SWL*: 220 kg / 485 lbs

Weight: 2.6 kg / 5.7 lbs

Material: HDPE



CURVED GLIDEBOARD

Order No. 476.0012

UDI/GTIN 7640144460945

The original GLIDEBOARD «**Banana**» sliding board is a very tried and tested classic from Switzerland, developed with therapists by SAMARIT and patented as a design. This sliding board is often used in particular to enable **sedentary patients** to **transfer independently** to **wheelchairs**, **car seats** or **chairs**.

Size: 68 x 23 cm / 27 x 9 inch

SWL*: 200 kg / 440 lbs

Weight: 1.2 kg / 2.6 lbs

Material: HDPE



SELFGLIDE

Order No. 476.0050

UDI/GTIN 7640144460952

This sliding board is used mainly by patients who can **transfer themselves** under **their own steam**.

Size: 50 x 21 cm / 20 x 8 inch

SWL*: 180 kg / 397 lbs

Weight: 0.8 kg / 1.8 lbs

Material: HDPE



TETRAGLIDE

Order No. 476.0863

UDI/GTIN 7640144460969

This sliding board has been **specially developed for paraplegic patients**. The carer can move the patient safely onto the sliding board **without lifting** thanks to its unique fish tail. It is characterised by the ease with which the patient can be sat upon, by the manner in which it can be **inserted from both sides** and is utilised for **transfer bed – chair – toilet, etc.**

Size: 63 x 23 cm / 25 x 9 inch

SWL*: 200 kg / 440 lbs

Weight: 1.1 kg / 2.4 lbs

Material: HDPE



Sit-On Monofil Sliding-Towel

Order No. 476.0200

UDI/GTIN 7640144460976

This Sit-On Monofil Sliding-Towel makes it **easy to slide onto the sliding board without clothing**. It is used by **healthcare professionals** and also by the **patients themselves**. Usually it is used with the Curved, Selfglide and Tetraglide. The monofil glide belt is suitable for **baths, showers** or the **toilet**.

Size: 30 x 45 cm / 12 x 18 inch

Material: Nylon fabric

Moving & Handling Slings

Handling slings are specially manufactured, upholstered cushion are for supporting a move.



PAT-SLING

PULL-UP SLING

Order No. 450.0195

UDI/GTIN 7640144461003

In the nursing sector, the **Pat-Sling** is a **versatile aid** and very **easy and safe in application**. Its permit utilisation for large as well as for slim patients (up to 120 kg).

Size: 20 x 50 cm / 8 x 20 inch

SWL*: 120 kg / 265 lbs

Weight: 0.5 kg / 1.1 lbs

Material: Nylon fabric reinforced with double hand grips



PULL-UP 6

LIFTING AND MOVING SLING

Order No. 450.5106

UDI/GTIN 7640144461010

The Samarit **Pull-Up 6 – Lifting and Moving Sling**

Lifting in the bed: The patient can easily be brought onto the Pull-Up by turning the patient onto his/her side. Two people can easily carry out the moving or lifting-up of the patient without the use of much strength. The grey, slide-resistant foam material serves for safe positioning of the patient.

Size: 50 x 150 cm / 20 x 59 inch

SWL*: 150 kg / 330 lbs

Weight: 1 kg / 2.2 lbs

Material: Nylong fabric reinforced, hand grips (6)



SLIDE-UP WIDE

MOVING GLIDE MATTRESS

Order No. 450.5110

UDI/GTIN 7640144461027

Samarit Slide-Up wide – **Moving Glide Mattress** possesses a wider upper portion which surrounds the upper body and can be held together by a nurse with one hand. This permits the **patient to be brought safely into a seated position and moved**.

Size: 100 / 50 x 100 cm / 39 / 20 x 39 inch

SWL*: 150 kg / 330 lbs

Weight: 1.6 kg / 3.5 lbs

Material: Nylon fabric with wide backpart to hold patient with one hand, two strong handles on each side.



GLIDEBELT

Order No. 450.2100

UDI/GTIN 7640144461041

Order No. 450.2200

UDI/GTIN 7640144461058

Order No. 450.2300

UDI/GTIN 7640144461065

GLIDEBELT helps to **position the slideboards under heavy patients**

Small 72 - 90 cm / 28 - 35 inch

Medium 88 - 106 cm / 35 - 42 inch

Large 104 - 122 cm / 41 - 48 inch



ONE-WAY SLIDE CUSHION

Order No. 450.4060

UDI/GTIN 7640144461034

The SAMARIT Seat One-way Slide cushion only **slides in one direction**. It allows healthcare professionals to pull a patient who has slipped forward back to the correct sitting position **with little effort**.

It is ideally suited for use in **rehabilitation clinics, care homes and care at home**.

Size: 45 x 42 cm / 18 x 17 inch

Weight: 0.8 kg / 1.8 lbs

Material: Tissue with one-slide, Top PU-textile grey coating

Turnplate / Turn cushion / Transfer sheets



Nursing aids for professionals

**Aid for controlled rotation, standing and seated patient transfer
in clinics, nursing homes and care homes.**



TURNPLATE 30

Order No. 480.0100


UDI/GTIN 7640144461089

The SAMARIT Turnplate is ideally suited to a **controlled rotation of the patient when standing up.**

Size: Ø 30 / 40 cm, Ø 12 / 16 inch, 8 mm / 0.3 inch high only

Weight: 0.5 kg / 1.1 lbs

Material: ABS / Teflon, blue

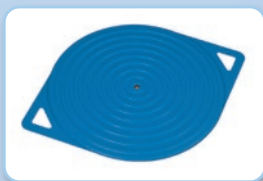

TURNPLATE 38

Order No. 480.0380

UDI/GTIN 7640144461072

The safest way to rotate a patient is when the helper puts his right foot in the middle of the Turnplate and the patient places his feet to the right and left of it.

Size: Ø 38 / 49 cm, Ø 15 / 19 inch, 8 mm / 0.3 inch high only

Weight: 0.6 kg / 1.3 lbs

Material: ABS / Teflon, blue

Limited stock

NEW

Full
non slip
coated!



BIG-TURNPLATE

Order No. 480.0400

UDI/GTIN 7640144461096

The SAMARIT BIG bariatric Turnplate is particularly suitable for a **controlled rotation of overweight patients.** The safest way to rotate a patient is when the helper puts his right foot in the middle of the turn plate and the patient places his feet to the right and left of it.

Size: Ø 41 / 54 cm, Ø 15 / 21 inch, 7 mm / 0.28 inch high only

Weight: 0.8 kg / 1.7 lbs

Material: ABS / Teflon, blue, full-scale anti-slip coating



SEAT & CHAIR TURN CUSHION

Order No. 480.1038

UDI/GTIN 7640144461102

The SAMARIT Turn Cushion is often used to **rotate a patient so they can sit on a car seat, a chair or an armchair.** It makes it easier to swing the legs round and thus makes it possible to stand up safely from the sitting position.

It is ideally suited for use in the **domestic sector.**

Size: Ø 38 cm / Ø 15 inch

Weight: 0.12 kg / 0.26 lbs

Material: Antimicrobial surface, Biocompatibility DIN 10993-5, Flameretardant, blue

The SAMARIT turnplates are characterized by **handiness, easy cleaning, safety and longevity.**



MOBICARE



MINICARE



Therapeutic Bed Toppers

The Samarit Mobicare Bed Topper has been developed to ease the **nursing of longterm bedridden patients** and is manufactured from **non-irritant** and **fire retardant materials**. Mobilise / turn patient easy to avoid pressure source.

The **mobile** Samarit Minicare Bed Topper cushion is **small, handy and universal to use**. The Minicare is used for the gentle **moving and positioning in the bed**. Patients who can no longer **move themselves** without assistance are helped by Samarit Minicare to greater mobility and independence.



MOBICARE

Mobilise / turn patient easy to avoid pressure source


Size: 90 x 195 cm / 35 x 77 inch

SWL*: 150 kg / 331 lbs

Material: with one side incontinent-proof 200 g/m2 hollowfibre

Order No. 450.0195

UDI/GTIN 7640144460990

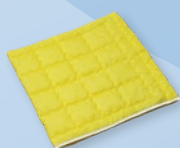

MINICARE

Mobilise in bed

Size: 60 x 60 cm / 24 x 24 inch breathable tissue

SWL*: 100 kg / 220 lbs

Material: Microfibre yellow

Order No. 450.0260

UDI/GTIN 7640144460983



CareGLIDE Model «1»

(Border red)

Order No. 450.0070

UDI/GTIN 7640144461112

1

Application: seated movements to the bedside, 90° rotation, small movements, sitting up in bed.

Size: 70 x 70 cm / 28 x 28 inch

Material: 2-fold silicone coated, PE fabric, skin-friendly, top slidable, washable 74°, latex-free



CareGLIDE Model «2»

(Border yellow)

Order No. 450.0100

UDI/GTIN 7640144461126

2

Application: right/left reclining movement, upper body on the side, turning patient into side position.

Size: 100 x 80 cm / 39 x 32 inch

Material: 2-fold silicone coated, PE fabric, skin-friendly, top slidable, washable 74°, latex-free



CareGLIDE Model «3»

(Border yellow)

Order No. 450.0130

UDI/GTIN 7640144461133

3

Application: right/left reclining movement of whole body, turning patient into side position, moving for replacing bed linen.

Size: 130 x 100 cm / 51 x 39 inch

Material: 2-fold silicone coated, PE fabric, skin-friendly, top slidable, washable 74°, latex-free



CareGLIDE Model «4»

(Border red)

Order No. 450.0180

UDI/GTIN 7640144461140

4

Application: right/left reclining movement, upper/lower sitting or reclining movement, transfer to bedside or wheelchair.

Size: 110 x 80 cm / 43 x 32 inch

Material: 2-fold silicone coated, PE fabric, skin-friendly, top slidable, washable 74°, latex-free



CareGLIDE Model «5»

(Border yellow)

Order No. 450.1310

UDI/GTIN 7640144461157

5

For heavyweight patients.

Application: Moving in bed to upper/lower reclining and sitting position, transfer bed / bed with the aid of 2 persons and pull sheet for very heavy patients, transfer / moving of spine damaged patients in bed.

Size: 130 x 100 cm / 51 x 39 inch

Material: 2-fold silicone coated, PE fabric, skin-friendly, top slidable, washable 74°, latex-free



CareGLIDE Model «6»

(Border yellow)

Order No. 450.7180

UDI/GTIN 7640144461164

6

For heavyweight patients.

Application: Transfer bed/bed with the aid of 2 persons and pull sheet for very heavy patients, transfer/moving of spine damaged patients in bed.

Size: 180 x 70 cm / 71 x 28 inch

Material: 2-fold silicone coated, PE fabric, skin-friendly, top slidable, washable 74°, latex-free



CareGLIDE Tubular Sheets for controlled move and rotation in bed.

CareGLIDE

MODEL 01 – 06 >

TRANSFER SHEETS / TUBULAR SHEETS FLAT SHEETS



CareGLIDE Moving Set

Flat Sheets / Tubular Sheet

Order No. 450.3160

UDI/GTIN 7640144461164

Moving Set consisting:

Size:

- 1 x 60 x 170 cm, 1 x 24 x 67 inch with handles
- 1 x 60 x 170 cm, 1 x 24 x 67 inch without handles
- 1 x 55 x 65 cm, 1 x 22 x 26 inch, tube

Material: 2-fold silicone coated, PE fabric, skin-friendly, top slidable, washable 74°, latex-free

CareGLIDE Flat Sheet

Order No. 450.3162

UDI/GTIN 7640144461171

Order No. 450.3163

UDI/GTIN 7640144461188

Size:

- 1 x 60 x 170 cm, 1 x 24 x 67 inch **with handles**
- 1 x 60 x 170 cm, 1 x 24 x 67 inch **without handles**

High quality standards
SAMARIT transfer aids have:

- Design patents
- 2 years warranty

SAMARIT
safe patient moving & handling

Samarit Medical AG

Gewerbestrasse 12

CH-8132 Egg, Schweiz

Tel.: +41 44 918 10 11 Fax: +41 44 918 24 39

e-mail: info@samarit.com - www.samarit.com

www.samarit.com

CE

