

BiGopsy[®] set up: 3 simple steps.

After backloading BiGopsy through the working channel of the ureterscope, secure the handle to the forceps according to the instructions as follows. To properly grip handle, place the thumb of your dominant hand into the thumb ring and control the spool between the index finger and middle finger on the same hand.



- 1** Use a finger from your free hand to depress the button and with your dominant hand retract the spool over the button. Secure handle in this position by releasing the button.



- 2** With your free hand, feed the forceps shaft through the handle until it passes completely through the viewing window of the handle.



- 3** Press button to release finger spool while firmly holding the spool in place, and then ease it into its original position. This secures the forceps shaft in the handle.



BiGopsy[®]
BACKLOADING BIOPSY FORCEPS

URO-BM-3SSBIG-EN-201002